

Instructions for use



1- Hold respirator in your hand as shown, with the nosepiece at your fingertip and the headbands hanging freely below your hand. Inspect to make sure that there are no pin holes in the staple locations.

2- Place respirator under your chin with the nosepiece up. Pull the top headband to the top back of the head.

3- Pull the bottom headband overhead and place around neck below ears.

4- Adjust headbands for comfortable fit. Push the nosepiece inward with your fingertips to conform to the shape of your nose.

5- Perform a fit check. Place both hands completely over the respirator and exhale. If air leaks around the nose or edges, adjust the nosepiece and/ or headbands until a good fit is achieved.

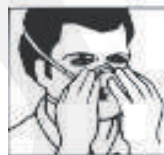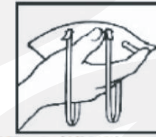

Service time limitation

- 1.- Do not use the same respirator for more than 8 hours, continuous or intermittent, in dirty conditions that could result in high filter loading.
- 2.- Leave contaminated area and remove respirator if breathing becomes difficult or proper fit cannot be obtained or the respirator is damaged, soiled, or distorted.

Cautions and limitations

This respirator does not supply oxygen. Not for use in atmospheres containing less than 19.5

Not for use in atmospheres immediately dangerous to life or health.

Do not exceed maximum use concentrations established by regulatory standards.

Failure to properly use and maintain this product could result in injury or death.

This respirator helps protect against solid particles that do not contain oil.

Never substitute, modify, add, or omit parts.

Refer to User's Instructions for information on use of these respirators.

Do not alter, wash, abuse or misuse this respirator.

Do not use with beards or other facial hair or other conditions that prevent a good seal between the face and the sealing surface of the respirator.

When stored in accordance with temperature and humidity conditions specified the product may be used until the "use by" date specified on packaging